CellSafe+ Biopsy Capsule

EBE-0201-02A, EBE-0201-03A, EBE-0301-02A, EBE-0301-03A

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

CellPath Ltd
80 Mochdre Enterprise Park,
Newtown, Powys, SY16 4LE, UK
T: +44 (0)1686 611333
E: sales@cellpath.com
cellpath.com

CellSafe+ Biopsy Capsule

EBE-0201-02A, EBE-0201-03A, EBE-0301-02A, EBE-0301-03A

A small device intended to be used in the laboratory to support / contain clinical tissue specimens (e.g., biopsy samples) to facilitate specimen processing [e.g., fixation (encapsulation in paraffin wax), dehydration, infiltration] in preparation for subsequent cytological or histological examination, and storage. It is in the form of a synthetic polymer container, with or without a lid, with drainage holes to maximize fluid exposure during tissue submersion in a parent device into which the container is loaded. This is a single-use device.

For laboratory use.

Store in a dry place. Don't store in direct sunlight.

This product does not expire under normal environmental conditions.

Processing Limitations: Suitable for high temperature processing. Suitable for microwave processing.

Instructions For Use

BIOPSIES PLACED DIRECTLY IN CELLSAFE+:

1. Place the biopsy into the CellSafe+, ensuring that it is $>220\mu\text{m}$. A maximum of two biopsies should be placed in a CellSafe+ at any one time.
2. The CellSafe+ can only be closed in one direction. When closing the CellSafe+ apply pressure evenly. Do not apply pressure directly to the mesh area; this may result in a lattice/ grid effect being embossed on the tissue sample.
3. Place the CellSafe+ in a fixative ensuring it is fully submerged.
4. Transfer the pot to the laboratory for processing, if applicable.

NOTE: The CellSafe+ is x-ray transparent.

5. Remove CellSafe+ from pot and place in cassette.

STRAINING TISSUE FRAGMENTS:

1. Pre-soak the CellSafe+ in ethanol or formaldehyde.
2. Ensure the fragments are $>220\mu\text{m}$.
2. Insert the larger plastic frame into a cassette.
3. Hold or support the cassette horizontally and slowly pour the suspension through the CellSafe+.

4. The tissue fragments will be retained in the CellSafe+, when all the suspension has been filtered through the CellSafe+, the CellSafe+ can be closed.

5. When closing the CellSafe+ apply pressure evenly. Do not apply pressure directly to the mesh area; this may result in a lattice/ grid effect being embossed on the tissue sample. Close the cassette.

PROCESSING THE CELLSAFE+ IN A CASSETTE:

If you are using any unusual reagents, we suggest that you carry out preliminary chemical compatibility tests before using CellSafe+ to contain tissue samples.

After processing, the CellSafe+ can be opened by pulling the 2 tabs away from each other and the tissue removed. Dispose of CellSafe+ in accordance with laboratory guidelines.

DO NOT REUSE THIS PRODUCT.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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