Instruction manual

Histology water bath W20

REF 720-2423

Instrucion manual (GB)	2-5
Instructions pour l'utilisation (FR)	6-9
Gebrauchsanweisung (DE)	10-13
Istruzioni per l'uso (IT)	14-17
Instrucciones para el uso (ES)	18-21
Instruções para el uso (PT)	22-25

Please follow the guidelines below, and read this manual in its entirety to ensure safe operation of the unit.

The unit can have hot surfaces and hot water.

Do not use the unit in an explosive environment

Avoiding Electrical Shock

Make sure that the voltage rating on the unit corresponds to the mains power voltage.

Make sure the mains power socket (wall socket) has a functioning electrical ground connection 

Make sure the units power plug corresponds to the mains power socket.

Always use the units grounded power cord between the unit and mains power socket.

Do not use the unit if the power cord is damaged or if the unit is damaged.

Do not open the unit, all repairs should be done by an authorized technician.

Compliance with local laws and regulations

The customer is responsible for applying for and obtaining the necessary regulatory approvals or other authorisations necessary to run or use the Product in its local environment. VWR will not be held liable for any related omission or for not obtaining the required approval or authorisation, unless any refusal is due to a defect of the product.

Equipment disposal

This equipment is marked with the crossed out wheeled bin symbol to indicate that this equipment must not be disposed of with unsorted waste.

Instead it's your responsibility to correctly dispose of your equipment at lifecycle -end by handing it over to an authorized facility for separate collection and recycling. It's also your responsibility to decontaminate the equipment in case of biological, chemical and/or radiological contamination, so as to protect from health hazards the persons involved in the disposal and recycling of the equipment.

For more information about where you can drop off your waste of equipment, please contact your local dealer from whom you originally purchased this equipment.

By doing so, you will help to conserve natural and environmental resources and you will ensure that your equipment is recycled in a manner that protects human health.

Thank you

Description	Quantity
Water Bath W20	1
Power cord	1
Instruction manual	1

Installation

Unpack the delivery. Check that the unit is not damaged and that all parts are included.

Place the unit on the work area. Connect the unit to an electrical grounded mains power socket, use the units power cord (which has a ground connection).

Check that the unit is OFF; no lights on the power switch or display. (If the unit is ON, green light on the power switch and red light on the display, then press the down side of the power switch, the lights goes OFF.)

Intended use and overview


The product is for use at a pathology/histology laboratory for in vitro diagnostics, for preparation of human tissue sections on glass slides for microscopy. It must only be used by suitably trained personnel. The units casing is made of white powder coated aluminium. The water bath (inside) is coated with black corrosion resistant thermoplastic. The unit is equipped with a power switch with green light, a temperature control unit with digital display, internal heating element and an overheat protection fuse.

The flat black surface is heated and can be used for drying and heating samples. The temperature is slightly lower than the water temperature.

An optional lid is available to cover the water. It prevents evaporation and dust contamination when the unit is not in use. VWR order number: 720-2424

Product Specifications

Mains power:	230 VAC 50/60 Hz
Power consumption:	260 W
Temperature range:	+20 to +70°C
Thermostat:	Digital
Temp. accuracy:	±1°C
Water capacity:	1,5 L
External dimensions:	W:247 x D:268 x H:78 mm
Internal dimension:	W:198 x D:156 x H:60 mm
Weight:	2,4 kg

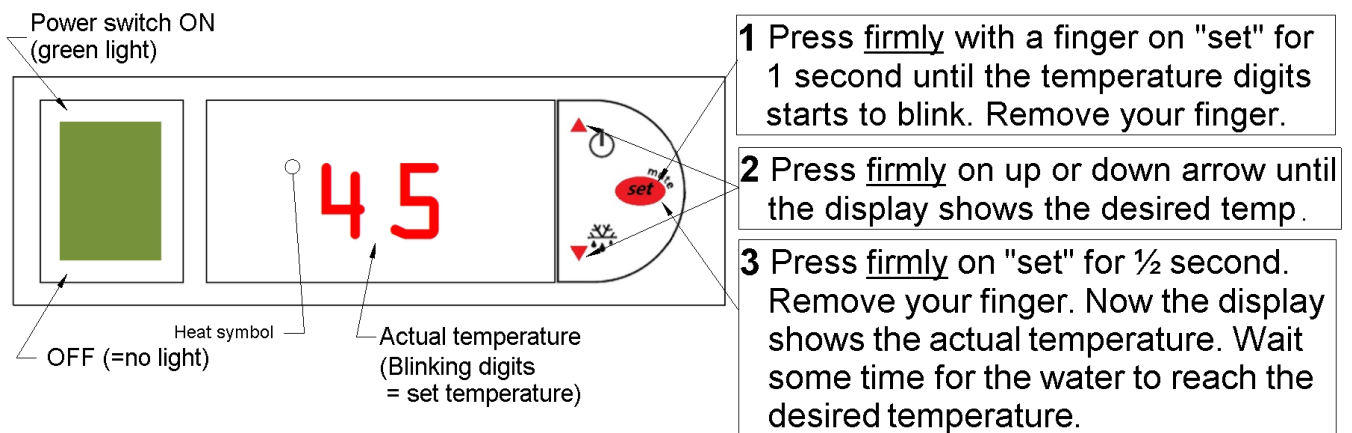Getting Started

Fill the water bath with 1,3 litres of water (distilled water to avoid deposits).

Turn the unit ON by pressing the upper part of the power switch; the power switch shows green light and the display shows actual water temperature.

To change the desired temperature, see instructions **1-2-3** in the figure above.

If you press longer than 3 seconds and the display shows **P5** then reset the unit by pressing the power switch OFF and then ON. Start at 1 again, see above.

Important Guidelines

Be careful when emptying the water bath. First turn the unit OFF. Then disconnect the power cord first from the mains supply and then from the unit. Please observe that the water can be hot. Carefully empty the water from the side of the unit. Do not let water enter the back or front of the unit. Wipe off the unit from any remaining water, leaving a dry surface.

Clean the unit with a soft cloth and use mild detergent or diluted (50%) alcohol.

Service and repair should only be made by an authorized technician.

Accessories

Description	Cat. No.
Lid. Covers the water, prevents evaporation and dust contamination	720-2424

Review the information in the table below to troubleshoot operating problems.

Problem	Cause	Solution
No heat	Too low set temperature	Increase the desired (set) temperature
	Heating element problem	Repair by an authorized technician
No power, no light	Power cord not firmly attached	Check the connection to mains power and to the unit
	The units fuse has blown	Replace the units fuse, located under the power cord socket
	The internal thermo fuse has blown	Repair by an authorized technician
Power switch has green light but no light on the display	Control unit not functioning	Repair by an authorized technician

Technical service

Web Resources

Visit the VWR website at www.vwr.com for:

- Complete technical service contact information
- Access to the VWR Online Catalogue, and information about accessories and related products
- Additional product information and special offers

Contact us For information or technical assistance contact your local VWR representative or visit. www.vwr.com.

Warranty

VWR International warrants that this product will be free from defects in material and workmanship for a period of two (2) years from date of delivery. If a defect is present, VWR will, at its option and cost, repair, replace, or refund the purchase price of this product to the customer, provided it is returned during the warranty period. This warranty does not apply if the product has been damaged by accident, abuse, misuse, or misapplication, or from ordinary wear and tear. If the required maintenance and inspection services are not performed according to the manuals and any local regulations, such warranty turns invalid, except to the extent, the defect of the product is not due to such non-performance.

Items being returned must be insured by the customer against possible damage or loss. This warranty shall be limited to the aforementioned remedies. IT IS EXPRESSLY AGREED THAT THIS WARRANTY WILL BE IN LIEU OF ALL WARRANTIES OF FITNESS AND IN LIEU OF THE WARRANTY OF MERCHANTABILITY.

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EU Declaration of Conformity

This declaration of conformity is issued under the sole responsibility of the following manufacturer:

KUNZ INSTRUMENTS AB

Stubbsundsvägen 17, SE-13141 Nacka, Sweden

SRN: not applicable yet

We hereby declare that the following product,

with intended use at pathology/histology laboratory for in vitro diagnostics, for preparation of human tissue sections,

Basic UDI-DI	Product code	Name
7350144587202423B3	720-2423	Histology water bath W20

comply with the following EU Regulation and Directive:

Regulation/Directive	Title
Regulation (EU) 2017/746	In Vitro Diagnostic medical devices (IVDR), class A non-sterile
Direktive 2011/65/EU	Restriction of Hazardous Substances (RoHS 2)

This declaration is supported by the use of these standards:

Standard	Title
IEC/SS-EN 61010-1	Safety requirements for electrical equipment for measurement, control and laboratory use Part 1: General requirements
IEC/SS-EN 61010-2-101	Safety requirements for electrical equipment for measurement, control and laboratory use Part 2-101: Particular requirements for in vitro diagnostic (IVD) medical equipment
IEC/SS-EN 61326-1	Electrical equipment for measurement, control and laboratory use - EMC requirements
IEC/SS-EN 61326-2-6	Electrical equipment for measurement, control and laboratory use - EMC requirements Part 2-6: Particular requirements - In-vitro diagnostic (IVD) medical equipment

Nacka 2024-06-04

Anders Olsson, Manufacturers representative

Kunz Instruments AB, Stubbsundsvägen 17, SE-13141 Nacka, Sweden

Local VWR offices in Europe and Asia Pacific

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India

VWR Lab Products Private Limited
No.139. BDA Industrial Suburb,
6th Main, Tumkur Road,
Peenya Post,
Bangalore, India – 560058
Tel.: +91-80-28078400
Email: vwr_india@vwr.com

Ireland

VWR International Ltd
Orion Business Campus
Northwest Business Park
Ballycoolin
Dublin 15
Tel.: +353 01 88 22 222
Email sales.ie@vwr.com

Italy

VWR International S.r.l.
Via San Giusto 85
20153 Milano (MI)
Tel.: +39 02-3320311/02-487791
Email: info.it@vwr.com

Korea

Avantor Performance Materials
Korea Ltd
2F ACE Gwanggyo Tower I
Daehak 4ro 17
Yeongtong-gu Suwon
Korea 16226
Tel.: +82 31 645 7256
saleskorea@avantorsciences.com

Middle East & Africa

VWR International FZ-LLC
DSP Laboratory Complex
125, Floor 01
Dubai, United Arab Emirates
Tel.: +971 4 5573271
info.mea@vwr.com

The Netherlands

VWR International B.V.
Postbus 8198
1005 AD Amsterdam
Tel.: +31 020 4808 400
Email: info.nl@vwr.com

Norway

VWR International AS
Brynsalleen 4
0667 Oslo
Tel.: +47 22 90 00 00
Email: info.no@vwr.com

Poland

VWR International Sp. z o.o.
Limbowa 5
80-175 Gdansk
Tel.: +48 058 32 38 210
Email: info.pl@vwr.com

Portugal

VWR International - Material de
Laboratório, Lda
Centro Empresarial de Alfragide
Rua da Indústria, nº 6
2610-088 Amadora
Tel.: +351 21 3600 770
Email: info.pt@vwr.com

Singapore

VWR Singapore Pte Ltd
18 Gul Drive
Singapore 629468
Tel: +65 6505 0760
Email: sales.sg@vwr.com

Spain

VWR International Eurolab S.L.
C/ Tecnología 5-17
A-7 Llinars Park
08450 - Llinars del Vallès
Barcelona
Tel.: +34 902 222 897
Email: info.es@vwr.com

Sweden

VWR International AB
Esbogatan 16
164 74 Kista
Tel.: +46 08 621 34 20
Email: info.se@vwr.com

Switzerland

VWR International GmbH
Lerzenstrasse 16/18
8953 Dietikon
Tel.: +41 044 745 13 13
Email: info.ch@vwr.com

UK-importer

VWR International Ltd
Customer Service Centre
Hunter Boulevard
Magna Park
Lutterworth
Leicestershire
LE17 4XN
Tel.: +44 (0) 800 22 33 44
Email: uksales@vwr.com