Export and Production
- since 1949 -

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APPLICATOR

STÄBCHEN

The applicator shaft of the swab is mounted on the tube plug which serves as an optimal grip for an efficient swab handling. Available in different versions on sampling site base:

- **Aluminium shaft, cotton or rayon tip:** atoxic shaft, with cotton or rayon tip, ideal for arduous or delicate sampling. The narrow dimension of the swabs shaft (only 0.9 mm), the high flexibility and strength, the small fiber tip, make it particularly suitable for urogenital, urethral, ocular, nasopharyngeal and paediatric uses.
- **Plastic shaft, cotton or rayon tip:** made in anti-shock polystyrene, very flexible but easily breakable as needed, with cotton or rayon tip, atoxic. Particularly suitable for mouth, throat, vagina, urogenital apparatus and skin.
- **Wooden shaft, cotton tip:** made in 100% cotton tip (European Pharmacopoeia) and atoxic wooden stick. Swabs for general purpose, suitable for throat, vagina and skin.

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TEST TUBE

RÖHRCHEN

Swab is contained in a cylindrical test tube Ø 12 x 155 mm, in transparent, unbreakable medical polypropylene (PP), to preserve sample and user's safety. With pressure cap in polyethylene (HDPE) with specific colour according to stick type. Pre-labelled on test tube to allow a correct sample identification. Item No., lot number and expiry date printed on each label for a full traceability.

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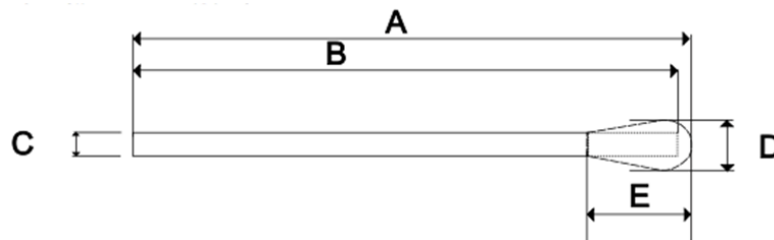
WOODEN STICK

Sterile swabs wooden stick and cotton tip, in polypropylene Ø 12 x 155 mm test tube with label. Available sterile in bulk or sterile individually wrapped.

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100% cotton tip (European Pharmacopoeia) and handle in biodegradable atoxic white birch wood, completely round, straight and polished, resistant to sudden changes of temperature. The tip is bonded to a handle, atoxic glue to fix cotton tip, and is perfect for general purpose medical use. Very absorbent, soft, and safe, tipped applicators perform consistently and reliably. Cotton tip can contain some substances that annoy some bacteria. The most sensitive bacteria may not survive after the collect. In these cases are preferable swabs with rayon tip.

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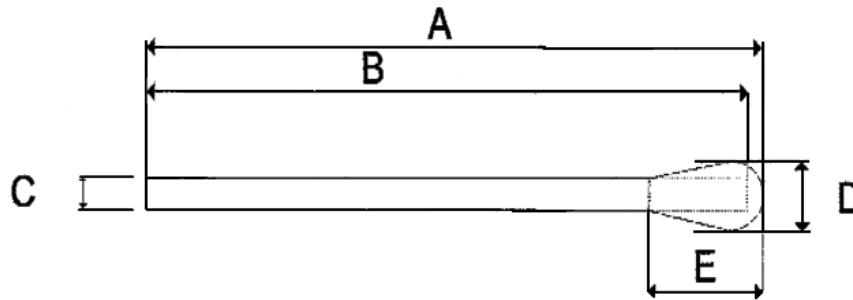
60 Monate

PLASTIC STICK

Sterile swabs plastic stick and cotton or rayon tip, in polypropylene Ø 12 x 155 mm test tube with label. Available sterile in bulk or sterile individually wrapped.

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This specialty swab is made with cotton or rayon fibers. It is bonded securely to a rigid white polystyrene handle by an aqueous based adhesive.

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Absorbance / Absorption: 100 µL ± 10% in 10 sec. (water + TSB) / (Wasser + TSB)

Dimensions / Abmessungen:

A: 150 ± 2 mm - **B:** 146 ± 1 mm - **C:** Ø 2.5 ± 0.1 mm - **D:** 5–6 mm - **E:** 15 ± 2 mm

Item No.	Type	Packaging
Art.-Nr.	Typ	Verpackung
Optional	Plastic stick (PS), cotton tip. In bulk <i>Plastikstab (PS), Baumwollkopf, lose</i>	Inner box / Innere Box: 100 pcs / Stück (180x115x135mm) External carton / Außenkarton: 1000 pcs / Stück (585x287x205mm)
1020002	Plastic stick (PS), rayon tip. In bulk <i>Plastikstab (PS), Viskosekopf, lose</i>	Inner box / Innere Box: 100 pcs / Stück (180x115x135mm) External carton / Außenkarton: 1000 pcs / Stück (585x287x205mm)
1020070	Plastic stick (PS), rayon tip. Individually wrapped <i>Plastikstab (PS), Viskosekopf, einzeln steril im Peel-Pack</i>	Inner box / Innere Box: 150 pcs / Stück (340x225x175mm) External carton / Außenkarton: 900 pcs / Stück (540x360x460mm)

Shelf life

60 months

Haltbarkeit

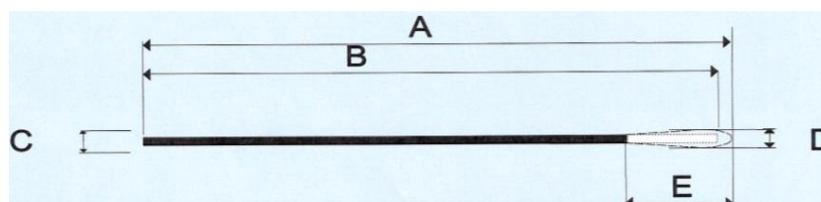
60 Monate

ALUMINIUM STICK

Sterile swabs aluminium stick and cotton or rayon tip, in polypropylene Ø 12 x 155 mm test tube with label. Available sterile in bulk or sterile individually wrapped.

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Swabs with cotton or viscose (rayon) tip, aluminium stick, atoxic, resistant to sudden changes of temperature. Atoxic glue to fix the tip. Very thin shaft and small tip for paediatric, urethral, urogenital and nasopharyngeal sampling. Without break point.

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Absorbance / Absorption: 20 µL ± 10% in 10" (water + TSB) / (Wasser + TSB)

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PRECAUTIONAL MEASURES

- The device must be suitable for both the selected transport medium (if present) and the defined sampling site.
- Use only for the collection and transport of human biological samples for the detection of potential pathogens such as aerobic, anaerobic and facultative anaerobic bacteria, compatible with the type of device selected.
- Do not deviate from the intended use.
- Do not use the product if it is expired or the package is opened/damaged. Sterility guaranteed if unopened.
- Use the device following aseptic procedures.
- Single-use device; do not reuse. Reusing the device could contaminate the sample and/or the patient.
- The fiber is only guaranteed to adhere to the shaft for instant sampling.
- The shaft is breakable, exert moderate pressure during sample collection.
- Store in a cool, dry place at a temperature between +2°C (35.6°F) and +30°C (86°F). Do not freeze.
- After use, the device may contain infectious microorganisms. Use appropriate PPE and dispose of the test tube and swab according to current regulations for medical waste.
- Keep out of flame or heat sources which might damage the product.
- Keep out of reach of children.
- Strictly follow the user's instructions.

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QUALITY ASSURANCE

All raw materials, components and every lot of finished product undergo strict quality inspections carried out by N.A. in order to check conformance to specifications.

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PRODUCT DETERIORATION

The content of the unopened and undamaged units is guaranteed to be sterile. Do not use the units in case of damage, dehydration or contamination. The Devices must be stored at +2°C (35.6°F) / +30°C (86°F) until used. Improper storage will result in a loss of efficiency. Do not use after expiration date indicated on label.

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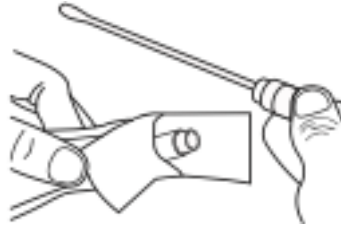
HOW TO USE

BENUTZUNG

If present, open the blister pack from the “peel here” indication. If the swab is in a test tube, remove the applicator from the test tube.

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Collect the sample from the concerned site.

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For a dry swab in a test tube, put the applicator into the tube, seal tightly with the cap and write the patient's data on the label.

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SPECIMEN CULTURES IN THE LABORATORY

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Manual Processing

Manueller Prozess

Remove the cap with swab applicator.

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Using the swab applicator, streak the agar plate while rolling the swab tip.

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