

Plum Eyewash - Product Information

Description

The Plum Eyewash bottles contains a 0.9 % sodium chloride solution that corresponds to the natural salt content of the eye fluid. This isotonic solution is suitable for rinsing the eyes as a first aid measure after an accident. The bottles are equipped with an ergonomic eye cup to ensure that the liquid can be applied correctly in an even and gentle flow. Squeeze the bottle lightly while rinsing. The DUO bottles are equipped with a special eye device that allows rinsing of both eyes at the same time. Our eyewash bottles are medical devices according to EU directive 92/42/EEC. A safety data sheet is available.



Range of application

Plum Eyewash is used for rinsing away foreign bodies from the eye (e.g. dust, dirt, metal and wood chips etc.) and also chemicals such as oils and solvents. The aim of immediate rinsing is to prevent the foreign bodies from sticking in the eye and causing serious harm. For caustic burns from by acids and alkali we recommend our sterile irrigation solution pH Neutral with 4.9% of phosphate buffer solution (product specification available) in order to achieve a neutralizing effect. After using pH Neutral we recommend to continue rinsing with Plum Eyewash 0.9 % sodium chloride until you reach a doctor. Note: Seek medical advice for all eye injuries!



Our eyewash plan informs you about how to rinse the eye in the event of different types of accidents:

EYEWASH PLAN

1	2	3	4
Determine type of substance	Select appropriate eye wash and rinse	Continue rinsing with Plum Eyewash	Rinse as directed
Foreign substances (e.g. dust, dirt, metal, wood, splinters, etc.)	Plum Eyewash	Plum Eyewash	Rinse until substance is removed
Acid	pH Neutral	Plum Eyewash	Rinse for about 2 minutes using pH Neutral and continue using Plum Eyewash until you reach a doctor or an emergency room
Alkali	pH Neutral	Plum Eyewash	Rinse for about 2 minutes using pH Neutral and continue using Plum Eyewash until you reach a doctor or an emergency room
Other chemicals (e.g. solvents or oils)	Plum Eyewash	Plum Eyewash	Rinse until you reach a doctor or an emergency room

IMPORTANT! Always ensure the eye wash liquid hits the eye with a soft and even flow. In case of concurrent injury on both eyes, use DUO for rinsing both eyes simultaneously.

Plum Eyewash - Product Information



Your benefits:

- High functional reliability - bottles are manufactured to DIN EN 15154-4 (non plumbed-in eye wash units)
- Flexible solutions - bottles in different sizes for mobile use and stations for mounting on the wall at the workplace
- Wide field of application (e.g. industry, laboratories, first aid kits, mobile use in vans, pockets etc.)
- Easy to handle - can be applied by the injured person without assistance
- Long shelf life of 3 years for sealed bottles
- No maintenance or replacement of the liquid required (as with ordinary water)

How to use the bottle:



Remove the bottle from the station



Twist the lid until the sealing breaks



Lean your head backwards and rinse



Or lean your head forward to avoid wet clothes

Instructions for handling:

Product for single use only. Do not use after expiry date (see the label). The sealing of the bottle must be intact to ensure safe rinsing. Once the sealing is broken there is a risk of infection if reused!

Bottle sizes:

Content	200 ml	500 ml	1000 ml	500 ml DUO	1000 ml DUO
Art. no.	4691	4604	4707	4861	4800
Rinsing time	about 2 Min.	about 5 Min.	about 10 Min.	about 2 Min.	about 5 Min.
Dimensions	H 165 x ø 55 mm	H 220 x ø 75 mm	H 270 x ø 85 mm	H 220 x ø 75 mm	H 270 x ø 85 mm