



TECHNICAL DATA SHEET

Medicom Steril-Peel Roll Stock

Product Description

Medicom Steril-Peel Roll Stock are manufactured from a medical grade paper that is thermally sealed to a laminated film. The reels contain process indicators used to indicate the pouches were processed via steam or EO sterilization. Available in multiple widths.

Intended Use

Sterilization reels or tubing rolls are intended to provide an effective method to enclose devices intended for sterilization in steam autoclaves and via ethylene oxide (EO). The process ink indicators of the reels are designed to indicate whether the pouch has undergone either steam or EO sterilization processes.

Standards of Conformance

ISO 11140-1	Sterilization of healthcare products – Chemical indicators
-------------	--


List of Items

221432	Medicom Steril-Peel Roll Stock, 2"x100'
221433	Medicom Steril-Peel Roll Stock, 3"x100'
221434	Medicom Steril-Peel Roll Stock, 4"x100'
221436	Medicom Steril-Peel Roll Stock, 6"x100'
221439	Medicom Steril-Peel Roll Stock, 9"x100'
221440	Medicom Steril-Peel Roll Stock, 10"x100'
221442	Medicom Steril-Peel Roll Stock, 12"x100'

Product Details	
Material Composition	Film laminate, medical grade paper, steam indicator ink, EO indicator ink, printing ink
Type	Single-use, Non-Sterile

Sterilization Cycle Parameters			
Sterilization Cycle Parameters	Steam Sterilization		Ethylene Oxide Gas
Time (min)	30	4	100-120
Temperature (°C)	121	134	50
Relative humidity (%)	N/A	N/A	60-85
Sterilant Concentration (mg/L)	N/A	N/A	600
Drying/Aeration time	15-30 min	20-30 min	12 hr

Product Dimensions			
Item Code	Product Dimensions		
	Width (mm)	Inner Width (mm)	Length (m)
221432	51	33	30.48
221433	77	59	
221434	102	84	
221436	152	134	
221439	228	210	
221440	254	236	
221442	305	287	

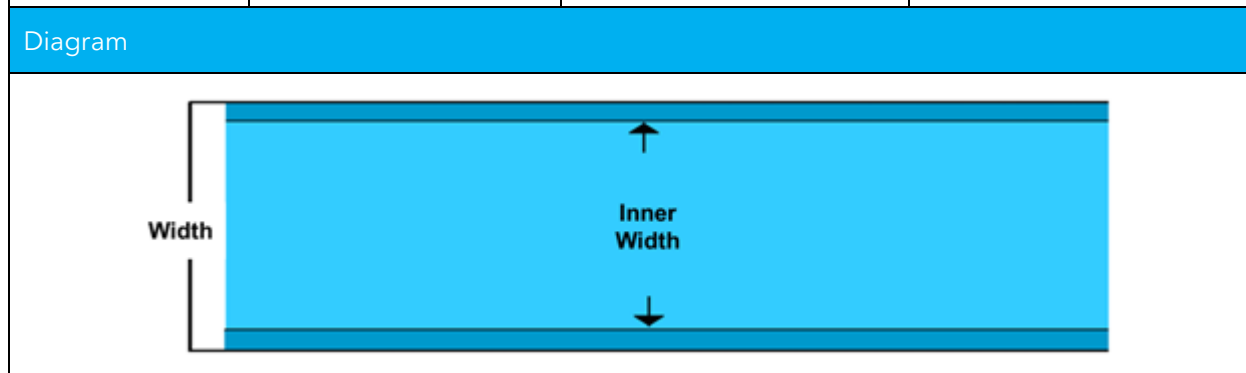

Process Indicator Color Change Guide



Additional Product Information

Packaging	10 Units/Case	
Post-Sterilization Shelf-Life	One (1) Year	
Market Classification	Canada	USA
	Class I Medical Device	Class II Medical Device