

Important

Please read these instructions before installing, maintaining, operating or using this equipment otherwise injury and/or damage may result.

Visually inspect equipment prior to putting it into use. Keep hands and clothing away from all moving parts. Always observe recommended weight capacity. If equipment is equipped with locking casters, be sure to lock all casters before using.

Cleaning Instructions

- All metal parts can be cleaned with a clean damp cloth. More stubborn stains may require a mild soap and water mixture. Dry completely using a soft cloth.
- To ensure years of lasting durability for your vinyl items, we suggest that you inspect your upholstery on a regular basis to ensure maximum upkeep.
- If a stain or discoloration should appear on your vinyl, clean immediately.
- To remove ordinary stains, we suggest you use a mild soap and water mixture. After cleaning, wipe soiled area clean with a damp cloth. Dry completely using a soft, lint-free cloth.
- For more difficult stains, we suggest a diluted mixture of water and 10% bleach. You should follow the general cleaning procedure with any of these special cleaners.
- As with any cleaning agents, you should use in an inconspicuous spot before using it on the original stain.

Your vinyl will resist, without serious discoloration, the repeated action of normal dilutions of chlorine bleaches or ammonia-based cleaners used for sterilization and cleaning in medical environments. **After any cleanser is used the vinyl surface should be wiped down with a clear water rinse.** This will eliminate any residue on the surface that could potentially have a negative reaction with the vinyl.