



PARASEP® SF & PARASEP® RANGE

FOR TRANSPORT AND CONCENTRATION OF FAECAL SPECIMENS FOR
INTESTINAL PARASITE EXAMINATION

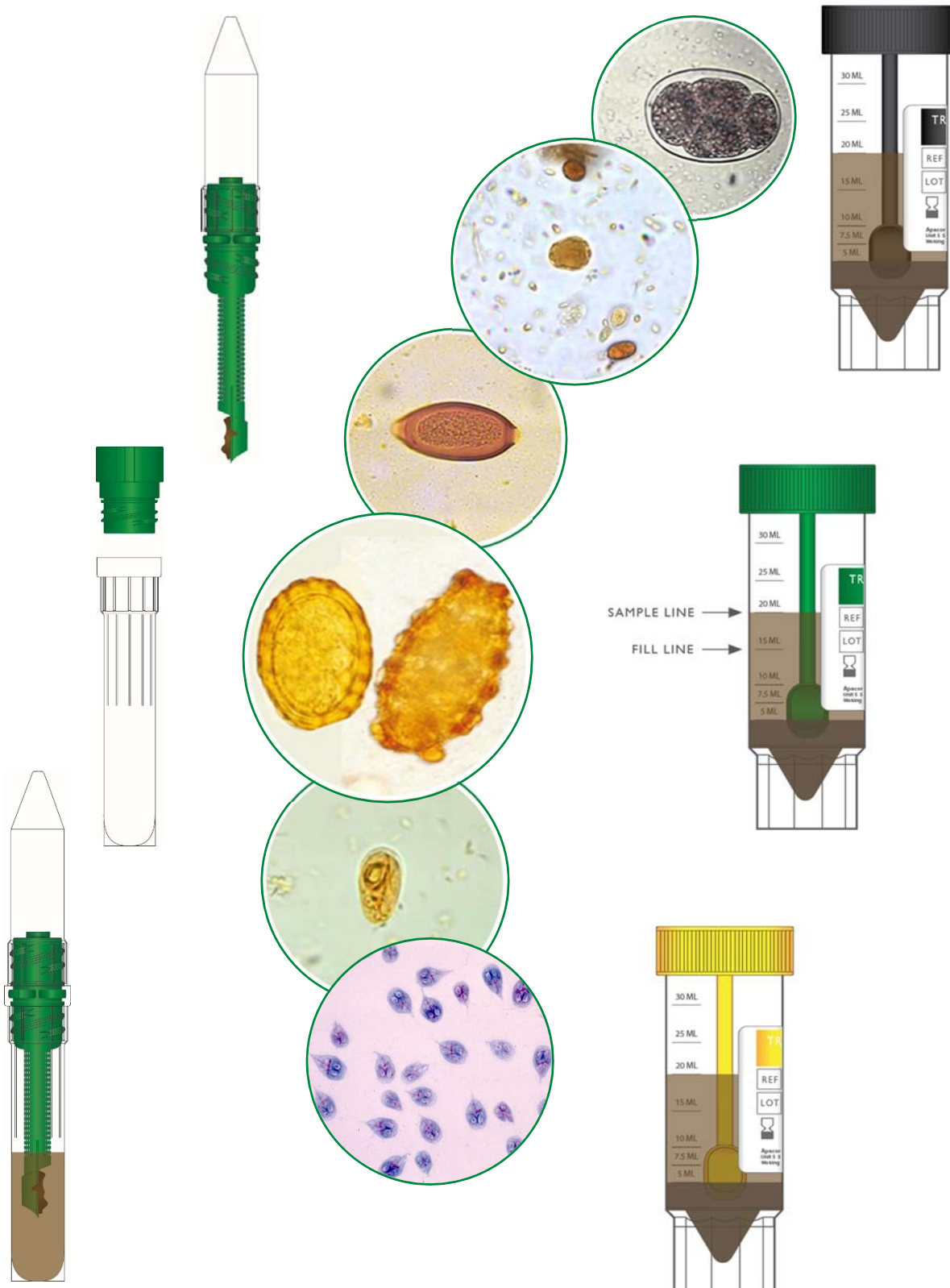


PARASITOLOGY

SINGLE USE IN VITRO DIAGNOSTIC DEVICE

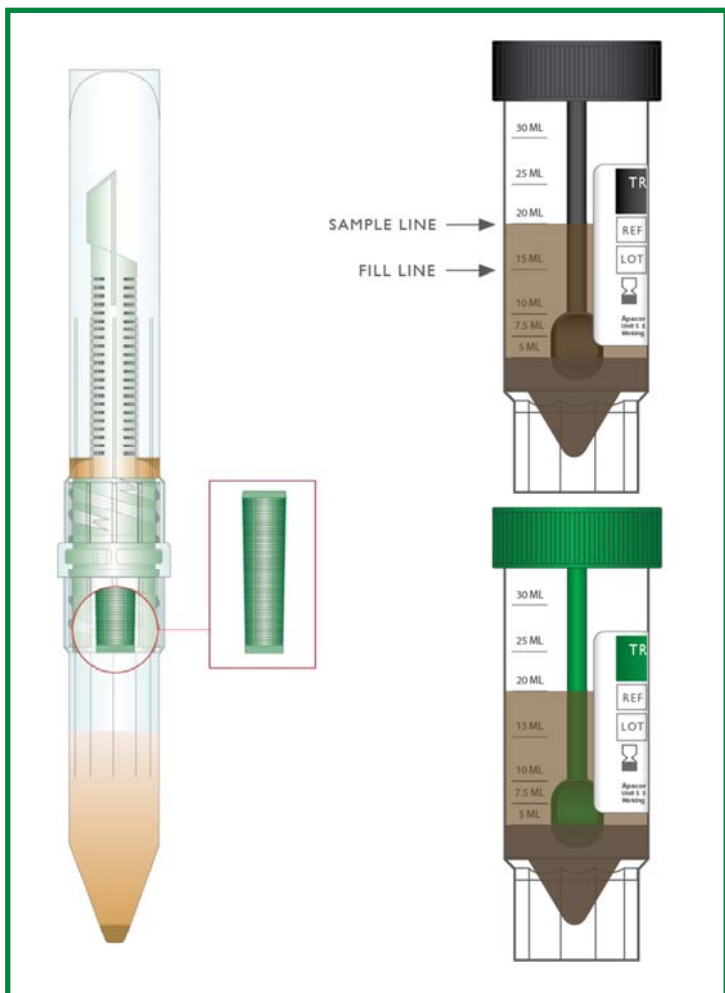

30ml Vial transfer to Parasep® SF

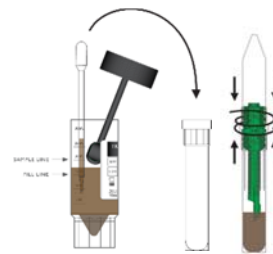

STEP 1 – TRANSFER 3 mL
SAMPLE INTO DEVICE



STEP 2 – CENTRIFUGATION



STEP 3 – DECANT SUPERNATANT



STEP 4 – DIAGNOSE SEDIMENT

ARUP Parasep® Procedure



STEP 1 – SAMPLES
RECEIVED BY LAB



STEP 2 – CENTRIFUGATION



STEP 3 – DECANT SUPERNATANT



STEP 4 – DIAGNOSE SEDIMENT

Benefits

- **Highest sensitivity** – patented vertical filter device removes blockage which occurs in all flat filter methods, thus increasing our yield of parasites
- **One vial required to conduct PCR, EIA, Lugol's Iodine wet mount and Trichrome stain** - patented formalin free solution enables lab not to purchase different vials for this and reduces costs to just one vial
- **Removes preparation steps**
- **Significant labor cost savings** - most efficient procedure as no re-suspending is required - 5 minutes saved on each sample
- **Lowest face value cost** – 25% less than the current market price for our versatile eco-friendly fixative
- **Enclosed system** – no contact for the lab personnel with stool or exposed fumes – no need for fume hood and no re-suspension
- **Enhanced sample clarity** – reduced amount of debris in sediment due to dual filter system

| Parasep® SF | | VWR Cat No | Parasep® | | VWR Cat No | Transport Vials | | VWR Cat No |
|---------------------------------|--|------------|--------------------|--|------------|-----------------------------|--|------------|
| SINGLE VIAL | | | SINGLE VIAL | | | 30ML TRANSPORT VIALS | | |
| Without reagent | | | Without reagent | | | Clean Transport Vial | | 10048-690 |
| 3.3 mL Formalin | | 10048-676 | Formalin | | 10048-638 | SAF | | 10183-870 |
| 2.7 mL Formalin | | 10048-678 | | | 10048-604 | AlcorFix™ | | 10184-190 |
| 3.3 mL AlcorFix™ | | 10144-672 | | | 10048-670 | | | |
| | | 10183-860 | | | | | | |
| 2 VIAL | | | | | | | | |
| AlcorFix™/Clean | | 10183-876 | | | | | | |
| | | 10183-868 | | | | | | |
| PATIENT CARE SINGLE VIAL | | | | | | | | |
| 8 mL AlcorFix™ | | 10147-167 | | | | | | |



1.800.932.5000 | vwr.com

Prices and product details are current when published; subject to change without notice. [Certain products may be limited by federal, state, provincial, or local regulations.] VWR makes no claims or warranties concerning sustainable/green products. Any claims concerning sustainable/green products are the sole claims of the manufacturer and not those of VWR International, LLC. All prices are in US dollars unless otherwise noted. Offers valid in US, void where prohibited by law or company policy, while supplies last. [VWR, the VWR logo and variations on the foregoing are registered (®) or unregistered trademarks and service marks, of VWR International, LLC and its related companies. All other marks referenced are registered by their respective owner(s).] Visit vwr.com to view our privacy policy, trademark owners and additional disclaimers. ©2015 VWR International, LLC. All rights reserved.